Little Sucker® NeoSucker®

REF Catalog No.

N204	N205TP	N224
N204C	N206	N224TP
N204TP	N207	N225
N205	N208	N225TP
N205C	N215	

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway , Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE
Westervoortsedijk 60
6827 AT, Arnhem, The Netherlands

MD Medical Device

Made in USA

D454 Rev M

English

INDICATIONS FOR USE
The Little Sucker/NeoSucker is indicated for use as a tip for suction tubing to aspirate fluids from the mouth and nares.

The Little Sucker Nasal Tip is indicated for use as a tip for suction tubing to aspirate fluids from the nares.

The Little Sucker Cover is indicated for use to protect the Little Sucker Oral and Nasal Suction Device.

DIRECTIONS FOR USE
Step 1 If cover in place (applies to N204C/N205C), unplug thumb port and slide suction tip out of cover.

Step 2 Connect barbed end of suction tip to suction line.

Step 3 Set suction to appropriate pressure per hospital protocol or physician recommendation.

Step 4 Block thumb port with thumb to begin suctioning.

Step 5 Suction mouth or nares with intermittent suctioning. Remove thumb from thumb port to stop suctioning. Block thumb port again to continue suctioning.

Step 6 Suction sterile water through suction tip after each use to clear debris.

Step 7 If applicable, reinsert suction tip into cover and insert port plug of cover into thumb port of suction tip.

Step 8 Replace suction tip/cover every 24 hours or per hospital protocol, whichever is sooner.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

For oral or nasal use only.

Inserting the suction tip too far into the mouth or nares can cause tissue trauma.

The suction tip/cover should remain free of debris.

The Little Sucker Cover is designed for use with the Little Sucker Oral and Nasal Suction Device specifically. It may not accommodate other suction devices. It is not intended for use with the Little Sucker 2-piece or Little Sucker Nasal Tip.

Single patient use only.

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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