# NEOTECH<sup>®</sup>

Products LLC

## NeoLead<sup>®</sup>

### ECG Electrodes

<b>REF</b>	Catalog No.	<b>DIRECTIONS FOR USE</b>
N300		<b>INSTRUCTIONS D'UTILISATION</b>
N301		<b>ISTRUZIONI PER L'USO</b>
N305		<b>INSTRUCCIONES DE USO</b>
N306		<b>INSTRUÇÕES DE UTILIZAÇÃO</b>
N315		<b>GEBRAUCHSANWEISUNG</b>
		<b>NÁVOD K POUŽITÍ</b>
		<b>ZALECENIA DOTYCZĄCE</b>
		<b>UŻYTKOWANIA</b>
		<b>إرشادات الاستخدام</b>

 Manufactured by  
NEOTECH PRODUCTS LLC  
28430 Witherspoon Pkwy, Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

 EMERGO EUROPE  
Prinsessegracht 20  
2514 AP, The Hague, The Netherlands

D484 Rev G

*English*  
**NeoLead<sup>®</sup>**  
**ECG Electrodes**

#### **INDICATIONS FOR USE**

The NeoLead<sup>®</sup> ECG Electrodes are intended for ECG monitoring. They are intended for use on pediatric (neonates, infants and children) patients.

#### **DIRECTIONS FOR USE**

##### **Preparation**

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Electrodes will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

##### **Application**

**Step 1** Before removing liner, warm the lead in hands for 60 seconds. May also be held under radiant warmer or heel warmer for 10 to 15 seconds.

**Step 2** Peel and discard liner.

**Step 3** Place the lead on skin and hold in place for 60 seconds to ensure proper adhesion.

##### **Removal**

**Step 1** Replace NeoLead every 5 days or per hospital protocol, whichever is sooner.

**Step 2** Saturate the lead with water or saline.

**Step 3** Slowly peel the lead away from the skin as you swab with water or saline.

#### **CAUTIONS & WARNINGS**

Federal law restricts this device to sale by or on the order of a physician.

NeoLead ECG Electrodes should only be used while patient is under continuous direct supervision of healthcare professionals.

Electrodes will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Do not use NeoLead (N300, N301) during x-ray or MRI procedures.

Do not use Radiolucent NeoLead (N305, N306, N315) during MRI procedures.

Discontinue immediately if skin irritation occurs.

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